

An Innovative Quick Test

Rapid Test for SARS-CoV-2 Total Antibody (COVID-19)



*Leader of diagnostic test for
infectious diseases*

Your Global Partner at IVD



Wantai's rapid SARS-CoV-2 total antibody test is designed for **evaluations of patients** suspected of infection with the SARS-CoV-2 Virus, an aid in the diagnosis of the **COVID-19** disease



Innovative



High
Performance



Easy to Use




Fast

Test Procedure


1. Test Preparation

Allow the test cassette to reach room temperature, use it within 20 minutes after opening the bag.

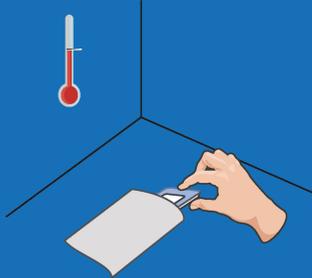


2. Add the sample

Add 10 ul of whole blood, serum or plasma samples. Add two drops of diluent buffer.

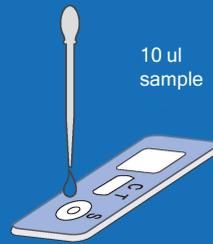

10 ul
sample

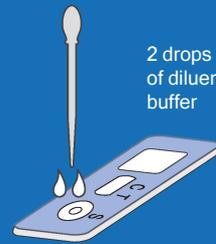


2 drops
of diluent
buffer

See product insert for detailed instructions.

3. Read the Result


Read the results in
15 min

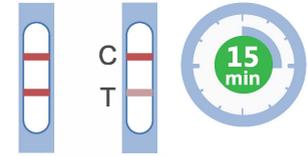

Positive



Negative

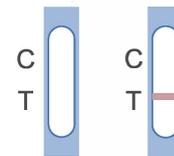

Invalid

Advanced Diagnostic Technology

01 Lateral flow test: suitable for rapid on-site testing.

02 Performance: sensitivity of 94,70% (125/132) and specificity of 98,89% (268/271). The test have been clinically validated during the COVID-19/2020 in China

03 Easy to use: without special equipment. Works with samples whole blood, serum or plasma.

04 Fast: clear answer in just 15 minutes

Order Information

Components	WJ-2710	WJ-2750
Cassette	x 10	x 50
Diluent	x 1	x 5

